# Should I Buy or Borrow a Pre-owned Breastpump?

Many mothers have asked if they can safely sell, purchase, or use a previously owned breastpump. Medela is concerned about the health and welfare of breastfeeding mothers and their babies. There is evidence that bacteria and certain viruses may be transmittable through breastmilk. For this reason, it is not advisable to use a previously owned, personal use breastpump, such as Medela's Pump In Style®. However, it is safe to use a rental pump, such as Medela's Symphony® Lactina®, or Classic™ pumps. The difference between personal use pumps and rental pumps is as follows:

## Personal Use Breastpumps

Personal use pumps that you buy at the store are personal care items, much like a toothbrush. Personal use pumps should never be resold or shared among mothers. The Medela Pump In Style® Advanced has an internal diaphragm that cannot be removed, replaced, or fully sterilized.

Pump In Style Advanced

Single Deluxe

Double Select

Therefore, the risk of cross-contamination associated with re-using a previously owned pump such as the Pump In Style cannot be dismissed, even when using a new kit or tubing. Similarly, the Medela Single Deluxe™ and Double Select™ Breastpumps have internal motors that cannot be removed, replaced or fully

sterilized. Another consideration when deciding to borrow or even lend a previously owned electric pump is the pump's motor life. A high quality electric double pump might last through the breastfeeding of your second child, or even several children. However, like computers or other electronic products, an electric breastpump has a limited lifetime. Medela guarantees its pump motors with a one year warranty. If you use an electronic pump that has been used for more than one year, there is no guarantee that it will generate as much speed and vacuum as it did earlier in its life. By using your own pump, you can compare the pump's performance with each

child. However, if you borrow a pump, you cannot gauge its performance to ensure it is operating at full capacity. Also, multiple women using these single-user products automatically voids the warranty of the Medela product.

## Rental Pumps


Rental pumps such as Medela's Symphony, Classic and Lactina are designed for multiple users.

These pumps have special barriers and filters to prohibit milk from entering the pump motor, which prevent cross-contamination. In


addition, each renter uses her own personal set of breastshields, containers and tubing, to ensure the safe use of these pumps.

Symphony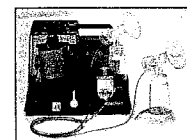

Classic

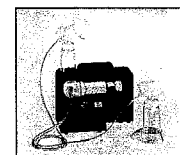Lactina

Different mothers have different pumping needs and economic means. For this reason, Medela pumps come in a variety of styles and prices. There are also grant programs available for mothers in need. For information on this and Medela products, ask your local rental station. For product information, availability and pricing, use our Breastfeeding National Network at **1-800-TELL YOU** to find a local rental/retail location. Medela Customer Service is available at **1-800-435-8316** for product questions.

**Contributions made by:** Kathleen B. Bruce, BSN, IBCLC, Barbara Wilson-Clay BS, IBCLC; Catherine Watson Genna, BS, IBCLC, and Mary Bibb, BA, IBCLC.

## References:

1 American Academy of Pediatrics: Report of the Committee on infectious Disease, ed 24, Elk Grove Village, Ill, 1997.

Blenkharn J: Infection risks from electrically operated breast pumps. *J. Hospit Infection* 1989; 13:27-31

Lawrence R and Lawrence R: *Breastfeeding: A Guide for the Medical Profession* 5th ed. Mosby, St Louis, 1999. Pg. 224-225.

Medela, Inc., P.O. Box 660, McHenry, IL 60051-0660 USA

Medela, Inc., 4090B Sladeview Crescent, Unit 2, Mississauga (Ontario) L5L 5Y5 CANADA

Phone: 1-800-435-8316 or 1-815-363-1166 Fax: 1-815-363-1246 [www.medela.com](http://www.medela.com) E-mail: [customer.service@medela.com](mailto:customer.service@medela.com)

Classic, Single Deluxe and Double Select are trademarks of Medela.

Medela, Lactina, Symphony and Pump In Style are registered trademarks of Medela.

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1547482 C 1104 ©2004 Medela, Inc. All rights reserved.
1547482 C 1104 ©2004 Medela, Inc. Todos los derechos reservados.